STERILE SWABS WITH AND WITHOUT TRANSPORT MEDIUM

Sterile, invasive sample collection system through direct contact with the patient. Intended for single use. Swabs sterilized by ethylene oxide.
 Transport media are medical devices for in vitro diagnostic (IVD), while the swabs, with or without media, are medical devices (MD) certified by CE0318.
 The handle is made of wood or plastic, and the tip is made of cotton or viscose.

Intended use:

The swabs with and without transport medium are sterile invasive products for collecting samples by direct contact with the patient, surgical type, with and without transport medium for subsequent microbiological analysis by trained healthcare personnel under appropriate hygienic conditions. The swab is intended to be used in clinical and hospital environments.

Presentation:

- Sterile swabs unitarily packed in peel-pouch bags or tubes.
- The swabs with transport medium are presented unitarily packed in a peelable peel-pouch bag, containing a swab and a tube with transport medium.

Warnings:

1. If it appears that the unitary packaging has lost its integrity, do not use the swab as there is no guarantee of its sterility.
2. Do not use in case of missing component or the product information is not displayed correctly.
3. Discard the product if you find any broken, dirt, chipped, loose tips and/or tipless swab.
4. Do not use if expiry date has been overcome.
5. Do not use if alteration is observed in the transport medium.
6. Do not ingest the transport medium under any circumstances.
7. For single use. The reuse of this product may cause infections.
8. Handle carefully. Any strong pressure during use, although unlikely, could break the swab. In case of damage, discard immediately.

9. Swab without transport medium do not require special storage conservation conditions. For swabs with transport medium, it is advisable to keep them in a dry place, protected from sunlight and at a temperature between 2-30°C.

General precautions:

1. Not suitable for any other application than its intended use.
2. For pediatric use it is recommended to use swabs with plastic handle.
3. The use of the swab presents no difficulty for the intended end users.
4. After use, the swabs must be decontaminated or treated as contaminated medical waste. It is the responsibility of each laboratory to handle, treat and dispose the waste according to the regulations. Unused swabs can be considered non-hazardous products and can be disposed of accordingly.
5. The swabs with transport medium include instructions with diagrams for use on the primary package (peel-pouch).

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